

## CR PHOSPHOR PLATE CLEANER (CRP- C)

The product offered is a solvent based formulation, which is mix of ethanol & derivatives, wetting agent and anti-static agents.

It is easy to use product, available in aerosol bottles and is supplied with good quality micro fibre cloths.

Dust and dirt gets deposited on the CR phosphor plates, which results into artifacts. To avoid deterioration of image quality, it is highly recommended to clean plates at regular intervals.

Use of cleaner will enhance life of a CR plates and will yield better quality of image.

CR phosphor plates are moisture sensitive. In case of plates being moistened or cleaned with water or water based cleaners, there are possibilities of loss in plate speed as well image quality problems.

The product is suitable for use with CR imaging plates only and should not be used for any other conventional plates.

Plates should be cleaned at least once a month. If dots or artifacts are observed in the Xray image, plate needs to be cleaned.

It is furthermore recommended to clean the interior of cassettes to avoid transfer of dirt or dust to the plate.



### Instruction for application:

- 1) Spray the cleaning solution onto the micro fibre cloth or soft, clean, lint free cloth  
(Do not pour the cleaning solution onto the plate)
- 2) Wipe the plate gently
- 3) Wait for few minutes to allow any excess cleaner to evaporate naturally.

### Box Contents:

1. 6 Bottles of 50 ml. Cleaner packed in aerosol bottle
2. 12 of micro fiber cloths approx. size of 15 x 18 cm

### Caution:

CR phosphor plate cleaner is solvent based and is inflammable in nature.

Do not store near fire. Preferable storage should be in an air conditioned area, at a temperature range of 20 to 30 degree Celsius.

It is recommended to take safety precautions like use of safety glasses and gloves.

This product is guaranteed against defect in manufacturing or packaging. Such warranty is limited to the extent of replacement of the defective goods only. No other warranty or liability expressed or implied.

Mfg. by: Rahul Photographic Company Pvt. Ltd.,

N-195-196, Tarapur MIDC Industrial Area, Boisar, Dist. Thane – 401 506. Maharashtra.

Customer Care No.: 09699847547

Email: [sales@rahulphotographic.com](mailto:sales@rahulphotographic.com)

Web: [www.rahulphotographic.in](http://www.rahulphotographic.in)